
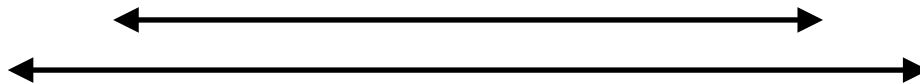


Zeiss SL-30 Slitlamp PD template.

2"

1.5" Circles

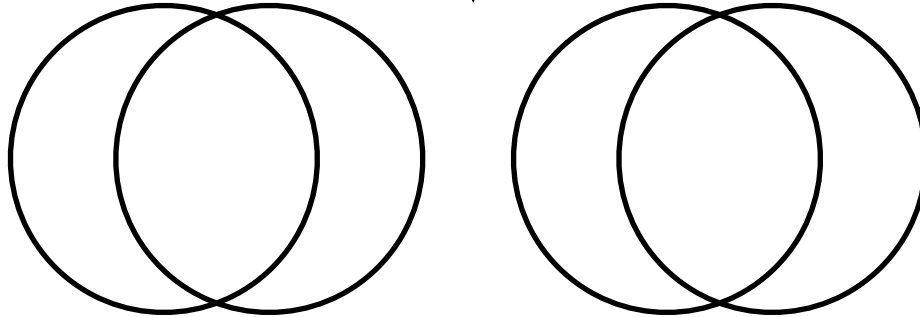

3 5/8" Narrow PD

4 3/4" Wide PD


**Covid protective shields**

**courtesy of Dr. Peter Van Houten, M.D.**

**Use this as a template to make slit lamp spray shields. Cut out the eye circles, place over a clear acetate. Draw the circles and cut out the circles on the acetate. Round the corners as shown and place the acetate over the eye pieces of each slit lamp or other equipment.**

**Stay Safe!**

**252-758-2402**

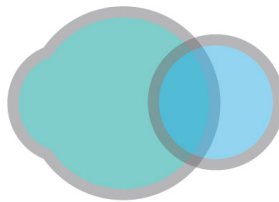

**East Carolina Retina**  
CONSULTANTS

252-758-2402  
2501A STANTONSBURG ROAD, GREENVILLE,  
NC 27834